# BATHTUB TRANSFER BENCH USER INSTRUCTIONS AND WARRANTY

G3-100KBX1, G3-100KBX2, G3-100KBPX1, G3-100KBPX2 : Weight Capacity 400 lbs (181 kg)  
G3-200KBX1, G3-200KBX2 : Weight Capacity 500 lbs (226 kg)

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**LIFETIME**  
LIMITED  
WARRANTY

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## **GENERAL WARNINGS:**

To ensure the safe assembly and operation of the product, these instructions must be followed. FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.

If you are unable to understand these warnings, cautions and/or instructions, contact a healthcare professional, dealer or Medline before attempting to use this product.

- DO NOT assemble and use this product without first reading and understanding this user guide, otherwise injury or damage may occur.
- DO NOT use if components are missing or are damaged, contact Medline at 1-800-Medline | 1-800-268-2848.
- DO NOT exceed weight capacity listed on the product.
- DO NOT make any adjustments while using the product.
- DO NOT attempt to use the arms to support full body weight.
- DO NOT stand on the product.

## **CAUTIONS:**

Users with limited physical strength and/or mobility should be supervised and/or assisted while using the product. Prior to use, ensure the following:

1. All push buttons are fully protruded and are properly engaged.
2. There are no loose, broken or missing components and ensure all parts are secure.
3. When using the backrest, ensure the product remains balanced. Excessive twisting or turning while seated on the bench may cause a loss of balance and/or damage to the product.
4. All four legs touch the floor simultaneously at all times.
5. The tips on the leg extensions do not contain any rips, wear or are missing. Immediately replace missing or damaged tips.

## ASSEMBLY:

1. Remove contents from box.
2. Lay transfer bench upside down on a flat surface.
3. Attach leg with the large suction cup (marked "A") by depressing the pushbutton and inserting into the receptacle tube marked "A" until the pushbutton snaps and locks into place. Repeat for second leg (Figure 1).

Figure 1

4. Attach leg with small rubber tip (marked "B") by depressing the pushbutton and inserting into the receptacle tube marked "B" until the pushbutton snaps and locks into place. Repeat for second leg (Figure 2).

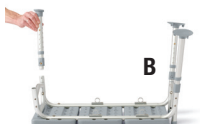
Figure 2

5. Turn transfer bench right-side-up. Depress armrest pushbuttons and slide into the armrest receptacle tubes with armrest arching outwards. (Figure 3).

Figure 3

6. To assemble backrest, insert pole into backrest support by depressing pushbutton and sliding into receptacle tube until it locks into place. Repeat for second pole (Figure 4).

Figure 4

7. Insert backrest into base by depressing pushbuttons and sliding supports through both receptacle tubes. **CAUTION:** Backrest must be locked and secured in each receptacle tube. (Figure 5).

Figure 5

**NOTE:** Back support is reversible to accommodate left or right tub entry. You can also change the seat depth of your transfer bench by pushing the pins under the seat and sliding the tubes forward until the pins lock into place.

8. Place legs with suction cups into bath tub.

## **Leg Height Adjustment:**

1. Before use, adjust the height of each leg to your specific needs, making sure all legs are at the same height. When your desired height is reached, lock legs into place by aligning buttons with holes until they “snap” into place.
2. Make sure legs are locked into place by pushing down or by pulling up on legs.

## **MAINTENANCE:**

Periodic visual inspection of this bench is recommended to ensure that all parts and hardware are secure; that components are in good working order and not worn, torn or loose; and that there are no obstructions or impediments to normal, safe operation.

If any part or mechanism appears to be unsafe or damaged, do not use the product. If you experience problems, please contact Medline at 1-800-MEDLINE | 1-800-268-2848.

## **CLEANING:**

- Clean with regular household cleaning products. Use a non-abrasive cleaner with warm water. Wipe dry after cleaning.
- DO NOT use abrasive powders, cleaners or scouring pads.

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## LIMITED WARRANTY

Your Medline product is warranted to be free of defects in materials and workmanship for the duration of use by the original purchaser. This device was built to precise standards and carefully inspected prior to shipment. This limited warranty is an expression of our confidence in the materials and workmanship of our products and assures years of convenient, dependable service. In the event of a defect covered by this warranty, we will, at our option, repair or replace the device. This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters and grips, which are subject to normal wear and replacement.

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---

**GUARDIAN**<sup>®</sup>

Medline Industries, LP,  
Three Lakes Drive, Northfield,  
IL 60093 USA.

---

Medline United States  
1-800-MEDLINE (633-5463)  
[www.medline.com](http://www.medline.com)  
[info@medline.com](mailto:info@medline.com)

Medline Canada  
1-800-268-2848  
[www.medline.ca](http://www.medline.ca)  
[Canada@medline.com](mailto:Canada@medline.com)

Medline Mexico  
01-800-831-0898  
[www.medlinemexico.com](http://www.medlinemexico.com)  
[mexico@medline.com](mailto:mexico@medline.com)

Some products may not be available for sale in Mexico or Canada. Please contact your Account Representative for additional information. We reserve the right to correct any errors that may occur within this manual. ©2021 Medline Industries, LP. Guardian and Medline are registered trademarks of Medline Industries, LP.

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