

ITEM #s: B2012 B2016 B2018 B2024 B2032

CHROME PLATED STEEL GRAB BAR W/ KNURLED GRIP



USER MANUAL - CHROME PLATED STEEL GRAB BARS

GENERAL EXPLANATION



- Professional installation required
 - Grab Bar (1)
- O [manum- 50
 - Screws (6)
- Knurled finish for a better grip
- 1 1/5" distance from wall
- Mounting hardware included
- +Supports up to 250 lbs
- Meets ADA & CSA Standards for Accessible Design

INSTALLATION INSTRUCTIONS

- Carefully read through all instructions before using and installing Grab Bars. This product requires professional installation.
- Check that there is no plumbing or electrical wiring in the installation area. Screws MUST be mounted into WOOD STUDS.
- Choose the Grab Bar location most convenient to user. May be placed vertically, horizontally or diagonally.
- Find wall studs and place grab bar on wall in line with studs and mark where screws will go in. After choosing a (vertical, horizontal or diagonal) position, drill holes for screws, ensuring the holes are in the stud. Use only wood screws for attaching to wood studs.

WARNING: Your professional contractor should use at least two wood screws in each flange to ensure that the grab bar is securely attached and strong. If it is not securely attached, using the grab bar may cause serious personal injury. Tools required for this step: Electric drill and masonry bit

After holes are drilled, place grab bar in position and attached with screws. You can also use toggle bolts for this step. If using toggle bolts (not included) ensure bolt is attrached before before putting it into hole. For best results, caulk the edges of the flange, to keep water and moisture out.

Using a screwdriver, tighten all screws and bolts. Periodocially, check screws are secure and tight. Do NOT use grab bar if it is not secure to the wall. This can result in serious personal injury.

USER MANUAL - CHROME PLATED STEEL GRAB BARS

GENERAL EXPLANATION



- Professional installation required
- Grab Bar (1)
- Screws (6)
- Knurled finish for a better grip
- +1 1/5" distance from wall
- Mounting hardware included
- Supports up to 250 lbs
- Meets ADA & CSA Standards for Accessible Design

INSTALLATION INSTRUCTIONS

- Carefully read through all instructions before using and installing Grab Bars. This product requires professional installation.
- Check that there is no plumbing or electrical wiring in the installation area. Screws MUST be mounted into WOOD STUDS
- O Choose the Grab Bar location most. convenient to user. May be placed vertically, horizontally or diagonally.
- Find wall studs and place grab bar on wall in line with studs and mark where screws will go in. After choosing a (vertical, horizontal or diagonal) position, drill holes for screws, ensuring the holes are in the stud. Use only wood screws for attaching to wood studs.

WARNING: Your professional contractor should use at least two wood screws in each flange to ensure that the grab bar is securely attached and strong. If it is not securely attached, using the grab bar may cause serious personal injury.

Tools required for this step: Electric drill and masonry bit

After holes are drilled, place grab bar in position and attached with screws. You can also use toggle bolts for this step. If using toggle bolts (not included) ensure bolt is attrached before before putting it into hole. For best results, caulk the edges of the flange, to keep water and moisture out.

Using a screwdriver, tighten all screws and bolts. Periodocially, check screws are secure and tight. Do NOT use grab bar if it is not secure to the wall. This can result in serious personal injury.

Rhythm Healthcare Clearwater, FL 33762

contactus@rhythmhc.com Website: www.rhythmhc.com (877) 843-6464

v3: 9/13/23