



User Instructions

Designer Folding Cane

Approx. user height:
4'11"–5'11"

Products for an
Independent Life

CONTACT US!
We're here to help

✉ info@novaoy.com
1800 557 6682
novaoy.com



THE NOVA PROMISE

1470 Beachey Place
Carson, CA 90746


How to Adjust Your Folding Cane

1. Depress the height adjustment pin that is visible in the series of holes for vertical height adjustment.
2. Lengthen or shorten cane to desired height.
3. Allow adjustment pin to pop back out to maintain desired height.
4. Ensure height adjustment pin is fully engaged before using.



How to Fit a Cane

1. Wear your regular walking shoes.
2. The top of the cane should be even with the break in your wrist and/or your hip joint.
3. Place the cane 6" in front of you and 6" over to the side. Your elbow should be bent at a 20 to 30 degree angle.

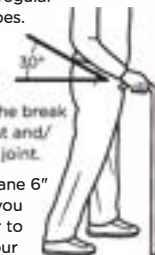

How to Walk with a Cane

1. Use the cane in the hand of the opposite side of your injured or weaker leg.
2. Stand in a triangle position. Place your good foot behind your body and your weak or injured foot in front of you opposite of the cane, forming the three points of a triangle. The toe of your weak/injured foot should line up with the cane.
3. Move the good foot first to the front of your body and maintain a triangle position. Then move your weak/injured foot and the cane together in front of your good foot. Make sure that the cane tip meets the floor flatly.
4. Take short steps to maintain your balance.



NOVA warrants this product to be free from defects in material or workmanship for a period of lifetime from date of purchase. If, within the specified warranty period, this product is proven to be defective, it will be repaired or replaced at NOVA's option. This warranty does not cover shipping costs incurred in replacement or repair of this product. NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This Lifetime Limited Warranty represents our confidence in the materials and workmanship of our products.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at our customer care number. Be prepared to indicate the nature of the defect, the product serial number, and the name and location of your dealer. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

This warranty does not cover problems due to user negligence or misuse of the product. This warranty shall not extend to non-durable components including, but not limited to, rubber tips, hand grips, casters, and other parts subject to normal "wear and tear".

THIS FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Lifetime Limited Warranty



Flashlight



Cane Clutch



Great Accessories for under \$20

Connect with us!

novajoy.com



novamedicalprod



novamedicalprod



novawalkerstalk



NOVA Medical Products

Quad Tip

- Allows your cane to stand upright
- Improves stability and traction



Cane Holder



Walker Skis

