Cast Protector User Guide

(8113-R, 8114-R, 8115-R, 8116-R, 8117-R, 8118-R, 8119-R)

Instructions

1. Check cast for sharp edges that may puncture or tear the cast protector.
2. Carefully place the flexible rubber seal over the cast/bandage and slowly pull the plastic ring up until the cast protector completely covers the cast/bandage. (Be sure the seal is secure around the limb to prevent any leakage.)
3. Be sure to dry the ring and seal off the cast protector before removing it from the limb.
4. Carefully place the flexible rubber seal over the cast/bandage and pull the plastic ring down until the cast protector is removed from the limb. (**DO NOT** pull on the clear plastic protector. This may cause tearing.)

Caution

Intended for single patient use.

Use this product only as a cast protector.

Be sure that the rubber seal is not too tight around the limb and does not affect blood circulation.

Cast protector can become slippery when wet. Use caution when showering or bathing.

Follow user instructions at all times.

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*****WARRANTY - Guaranteed to last the life of the cast or bandage*****

CONTACT US!
We're here to help

 info@novajoy.com

 1 800 557 6682

**Updated December 15, 2016*

Connect with us!
novajoy.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
YouTube: NOVA Medical Products