

journey solite[®] c2

PRODUCT MANUAL



SO LITE C2

 **journey**
HEALTH & LIFESTYLE[®]

www.shopjourney.com

1.	Preface	4
2.	Dos and Don'ts	5-7
	2.1 Open-package inspection	5
	2.2 Explanation of symbols	5
	2.3 Safety Instructions	5
	2.4 Intended Use	7
3.	Product Description	8-9
	3.1 Product Feature	8
	3.2 Components	8
	3.3 Technical Specifications	9
4.	Installation and Adjustment	10-14
	4.1 Unfolding and folding the wheelchair	10
	4.2 Unfolding and folding the backrest	11
	4.3 Assembling and removing the footrest	12
	4.4 Assembling and removing the rear wheels	13
	4.5 Assembling and removing the seat and backrest cushion	13
	4.6 Transporting carrying the wheelchair	14
	4.7 Lift back the armrest	14
5.	Use	15-19
	5.1 My operating instructions	15

5.2 Nursing Instructions	16
5.3 Braking by user	17
5.4 Braking by other person	17
5.5 First using wheelchair	18
5.6 Moving the user in and off the wheelchair	19
6. Maintenance	20-21
6.1 Cleaning the wheelchair	20
6.2 Disinfecting the wheelchair	20
6.3 Inspection, service and maintenance	21
6.4 Reusing	21
6.5 End of use	21
7. Warranty	22
7.1 Repair and replacement	22
7.2 Maintenance parts and period	22
7.3 Maintenance records	22
8. Transportation and Storage	23
8.1 Transportation and Storage	23

Welcome to purchase and use our manual wheelchair.

This product is specially processed from magnesium-aluminum alloy, which is an aviation material. It is with high strength and light weight, so it can be lighter and more comfortable for users.

Your safety is the most important for us. Before you use the wheelchair, Please read this instruction manual carefully, which plays an important role in your safe operation.

This instruction manual is one integral part of the product design. As the design changes, if some illustrations and figures in the instruction manual can not match the wheelchair you purchased. The manufacturer reserves the right to improve the product.

Hope the product can bring more convenience to your life. At the same time, we also look forward to your valuable suggestions on the quality and after-sales service of our wheelchairs. You can give us feedback by email, call or visiting our website (www.shopjourney.com).








Please keep the following information.

2.1 Open-package Inspection

Please check if the main frame and accessories are missing, damage or lack, please contact the dealer or the manufacture in time.

- Wheelchair — 1 piece
- Seat Cushion — 1 piece
- Back Cushion — 1 piece
- Footrest — 1 pair
- Anti-tip Wheel — 1 pair (only HM305-A1)
- Instruction Manual — 1 piece
- Product Qualified Card — 1 piece, Warranty Card — 1 piece



2.2 Explanation of Symbols

 Warning	Indicates that personal injury may result from misuse
 Caution	Indicates that personal injury or damage to scooter may result from misuse
 Mandatory	Indicates instructions must be followed
 Prohibited	Indicates something specific is not allowed
 Load Capacity	The max load capacity
 Reading The Instruction Manual	Indicates to read the instruction manual carefully
 Warning Hands Pinching	Mind your fingers


2.3 Safety Instructions



Warning!

-  **Prohibit:** The user is strictly prohibited from sitting in the wheelchair when wheelchair is in any moving vehicle.
-  **Prohibit:** It is strictly forbidden to brake at once during the wheelchair is running. If there is no caregiver, please hold the push rim and brake gradually.

- ⊘ **Prohibit:** It is strictly forbidden to stand on the foot plate, otherwise the wheelchair will tip over.
- ⊘ **Prohibit:** It is strictly forbidden for the user to modify the wheelchair without the consent of the technician. Unauthorized modifications can lead to accidents.
- ⊘ **Prohibit:** Please do not stick your body out during driving, otherwise it will affect the stability of driving. (Especially on steps and bumps).
- ⓘ **Mandatory:** Be careful when driving and stopping on sloped roads.
- ⓘ **Mandatory:** Users should be especially careful of cars and pedestrians at night and raining days.
- ⓘ **Mandatory:** Do not use the wheelchair on muddy or slippery roads, soft lawns, areas with ice and snow. Use a wheelchair with caution under the influence of alcohol or medication.
- ⚠ **Caution!**
- ⓘ **Mandatory:** Practice with caregiver before using wheelchair.
- ⓘ **Mandatory:** Please do not lean back in a wheelchair without anti-tipping.
- ⓘ **Mandatory:** Please do not tilt the wheelchair sideways and do not lift the wheelchair during the wheelchair is running.
- ⓘ **Mandatory:** Please park the wheelchair on a flat place.
- ⓘ **Mandatory:** Do not place tires near heating appliances.
- ⓘ **Mandatory:** Please do not hold the tube which on seat both sides when you press down to prevent your hand from being pinch, when unfolding the wheelchair.
- ⓘ **Mandatory:** The plates should be adjusted to a height of 2 inches above the ground.
- ⓘ **Mandatory:** The wheelchair must be properly installed with the anti-tipping before use. The ground height of anti-tipping should remain 1 inch. Please keep your feet on plates during running.
- ⓘ **Mandatory:** When going up the steps, the caregiver should step on the anti-tipping of the wheelchair, hold the handle with both hands to press down, then push the wheelchair forward to go up the steps.

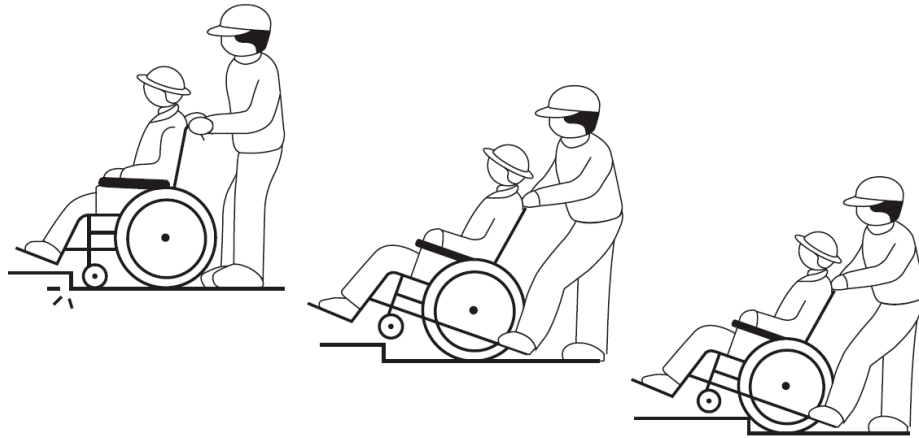

2.4 Intended Use

It is used for the disabled, the sick and the elderly and the infirm. Only 1 person can be carried and user's weight does not exceed 220 lbs.

It is forbidden for minors, mentally handicapped and other incapable persons who cannot use this wheelchair alone.

3.1 Product Feature

3.1.1 Wheelchairs can be operated by user or also can be assisted by family members, escorts or caretakers.

3.1.2 It is a standard wheelchair. The backrest has no function of angle adjustment. There is also no function for exercise and bathing, it is just only a standard wheelchair.

3.1.3 This product is mainly used in hospitals, nursing homes and households. Pay attention to driving on flat roads with small gradients and to driving on dry, water-free and ice-free roads.

3.2 Components

The wheelchair is composed of frame, armrest, seat, wheel, footrest.

3.2.1 Parts (fig 3-1)



- | | | | |
|----------------|----------------|------------------|----------------|
| 1. Backrest | 4. Fold Switch | 7. Rubber Sleeve | 10. Foot Plate |
| 2. Hand Grip | 5. Arm Pad | 8. Brake | 11. Footrest |
| 3. Brake Lever | 6. Rear Wheel | 9. Front Wheel | 12. Seat |

3.3 Technical Specifications

Model	08970 BLK/CFR
Overall Dimension (LxWxH)	42in x 27in x 36.5in
Folded Width	13 inches
Folded Height	29.5 inches
Seat Width	17.7 inches
Seat Depth	17 inches
Seat Surface Height at Front Edge	18.9 inches
Armrest Height	7.8 inches
Distance Between Armrests	19.6 inches
Backrest Height	16 inches
Foot Plate Height Above Ground	4.3
Total Wheelchair Weight	24 lbs
Loading Capacity	240 lbs
Front Wheel Dimension	6 inches
Rear Wheel Dimension	20 inches
Minimum Turning Radius	≤33

4.1 Unfolding And Folding The Wheelchair

4.1.1 Unfolding

4.1.1.1 When the brake is unlocked, stand in front of or behind the wheelchair, grab both sides of the armrest with both hands, and pull them away with force; (Fig 4-1)

4.1.1.2 Press both sides of the cushion with the palm of your hand until the cushion is flat; (Fig 4-2)

4.1.1.3 Open the foot plate.

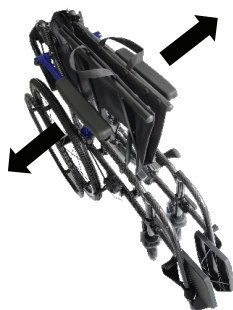


Fig 4-1



Fig 4-2



Caution: When unfolding, do not pull the armrest hard to avoid damage to various parts. When pressing down on the seat cushion, do not hold your fingers on the seat tube to avoid pinching your fingers.

4.1.2 Folding

First fold up the foot plate (C), grasp both sides of the seat sling (A) with both hands, then lift the seat from the center, and finally press the two sides together (B). (Fig 4-3)



Fig 4-3

4.2 Unfolding And Folding The Backrest

4.2.1 Unfolding

Hold the handle and lift up until the pin pops out. (Fig 4-4, Fig 4-5)



Fig 4-4



Fig 4-5

 **Caution:** Be careful not to pinch your hands.

4.2.2 Folding

One hand hold the handle and the other hand press down the folding switch (A) of the back folder to release the pin from the back folder. (Fig 4-6)

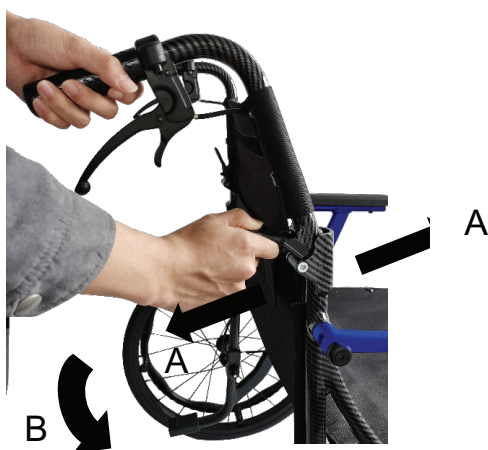


Fig 4-6

 **Caution:** Be careful not to pinch your hands.


4.3 Assembling And Removing The Footrests

4.3.1 Assembling

4.3.1.1 Insert the footrest shaft into the tube from the side.

4.3.1.2 Rotate the footrest inward until the pull pin is fully seated in the hole, you will hear a “click”.

- 1. Footrest Shaft
- 2. Tube
- 3. Pin
- 4. Hole
- 5. Pull Pin

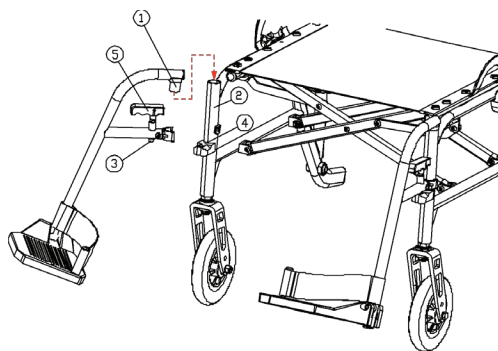


4.3.2 Removing

4.3.2.1 Pull the pull pin (⑤), Get the pin (③) completely out of the hole (④)

4.3.2.2 Rotate the footrest to the outside and lift it up to remove the footrest.


- 1. Footrest Shaft
- 2. Tube
- 3. Pin
- 4. Hole
- 5. Pull Pin

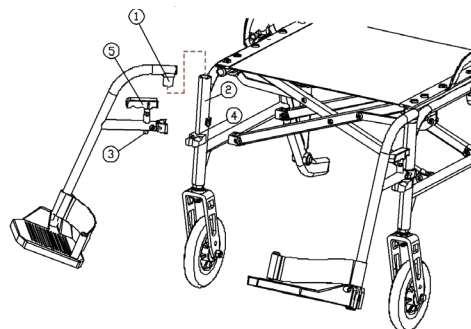

4.3.3 Adjusting foot plate

4.3.3.1 Please check that the foot plate height above the ground is more than 2 inches, adjust two sides of the plates to keep them with same height.

4.3.3.2 The foot plate can be adjusted in 3 steps using the included tools.

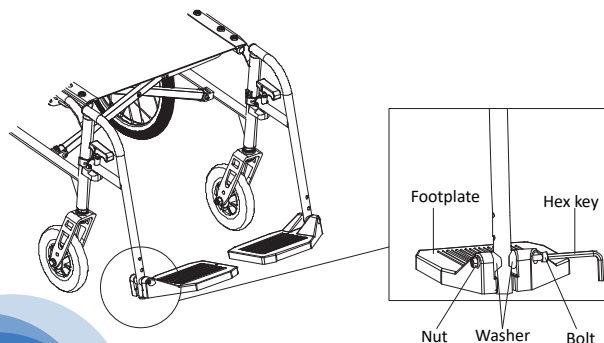


4.4 Assembling And Removing The Rear Wheels

4.4.1 Assembling

4.4.1.1 Press down the quick release button, then insert the quick release into the inner hold of the matched frame tube until the pin pops out.


1. Quick Release Button
2. Rear Wheel
3. Frame Tube

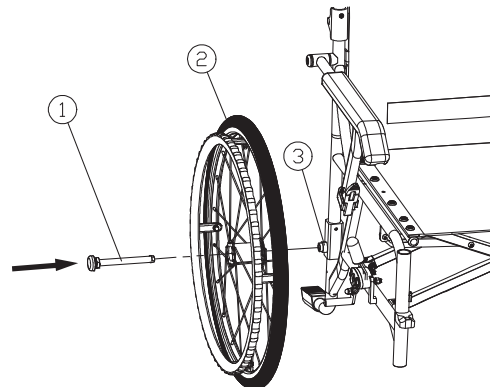

4.4.2 Removing the rear wheels

4.4.2.1 Press the quick release button, then pull out the rear wheel.

1. Quick Release Button
2. Rear Wheel

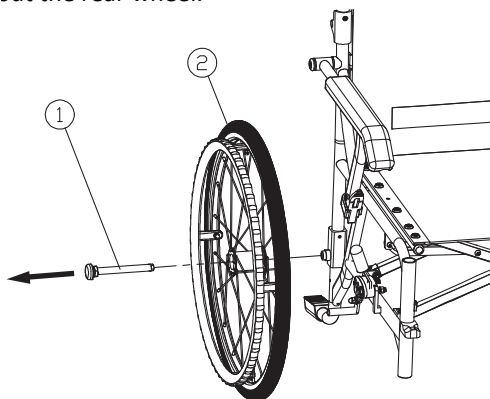

4.5 Assembling And Removing The Seat And Backrest Cushion

4.5.1 Tear the stickers of Velcro on seat and backrest cushions, then paste them into the corresponding places.



4.6 Transporting Carrying The Wheelchair

4.6.1 The best way to move a wheelchair is to use its wheels and push the wheelchair to move.

4.6.2 You can also lift the wheelchair by grasping the seat tube after folding the wheelchair.

4.6.3 If you need to put the wheelchair in the car, you can fold the wheelchair, remove the rear wheel and footrest, and put it in the trunk of the car.


4.7 Lift Back The Armrest

4.7.1 Lift back the armrest.

4.7.1.1 Press the armrest latch to release the armrest.

4.7.1.2 Lift the armrest back.

4.7.1.3 It is the same operation for the other side of armrest.

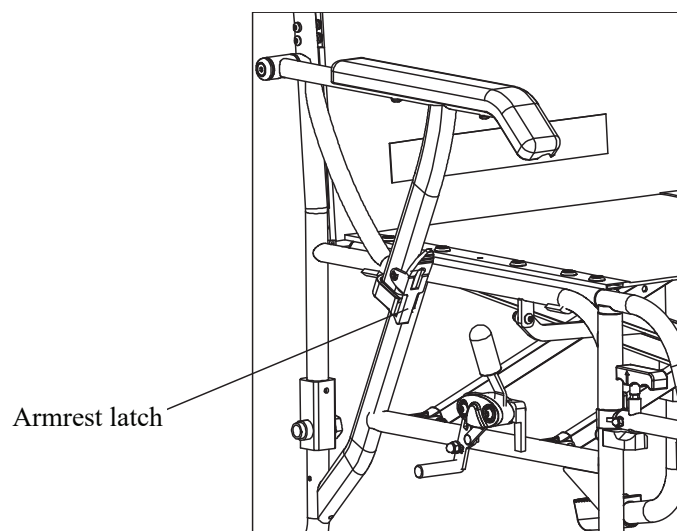


5.1 My Operating Instructions

5.1.1 You can drive normally by rolling the handrim on the sidewalk.




5.1.2 Go up the steps, stop in front of the steps, make the front wheel at a 90 degree right angle to the step, lean forward and push hand rim hard.

Please don't cross the steps higher than 1.5 inches.

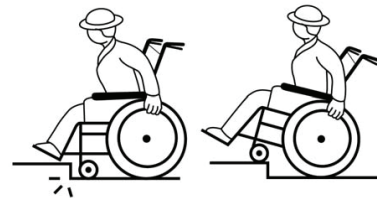

5.1.3 When going uphill, use a forward-leaning posture to climb forward; when going downhill, keep forward leaning and slide the hand rim in the opposite direction to go backwards. For very long steep slopes, Please do not go up and go down by yourself.

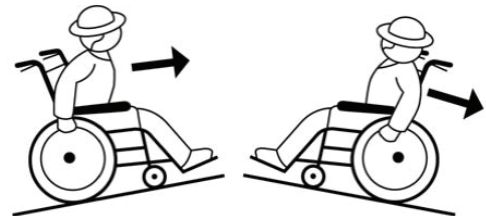

5.1.4 When there is a groove on the road, please cross the groove at a right angle to the direction of the groove. If the groove is too wide and too deep, please don't cross it yourself.




5.2 Nursing Instructions


5.2.1 On the sidewalk, you can hold two sides of handle and push the wheelchair forward.



5.2.2 When going up the steps, the escort should step on the anti-tipping of the wheelchair, hold the handle with both hands to press down, then push the wheelchair forward to go up the steps. If the wheelchair has been installed with anti-tipping, you can adjust the height of anti-tipping first, and then push the wheelchair in the above method.

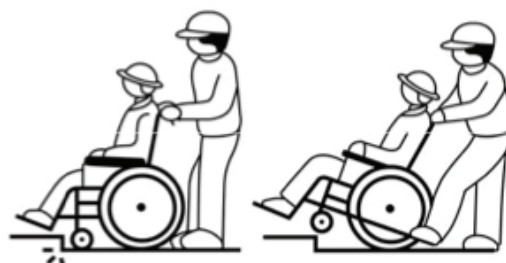


5.2.3 When going uphill, the pusher keep leaning forward, push the wheelchair uphill. When going downhill, keep the normal posture, and back in the opposite direction.

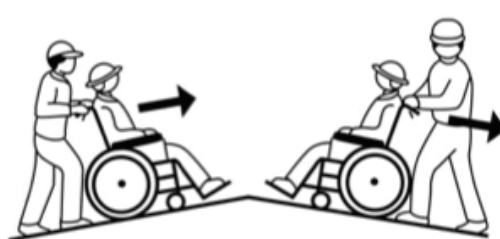

5.2.4 When there is a groove on the road, please cross the groove at a right angle to the direction of the groove.

5.3 Braking By User

5.3.1 Sitting in the wheelchair, holding the both sides of brake handle. Pull back to brake, the opposite direction is to release the brake.


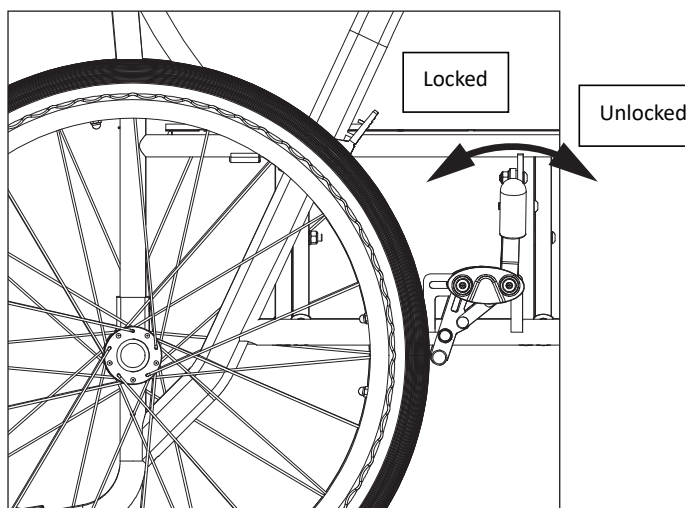


5.4 Braking By Other Person

5.4.1 Gently grip two sides of brake handles to brake. Release two sides of brake handles to push.

5.4.2 Firmly grip two sides of brake handles to brake, the wheelchair is in a state of parking brake, then you can release both of your hands.

5.4.3 Lift up to release the parking brake status.

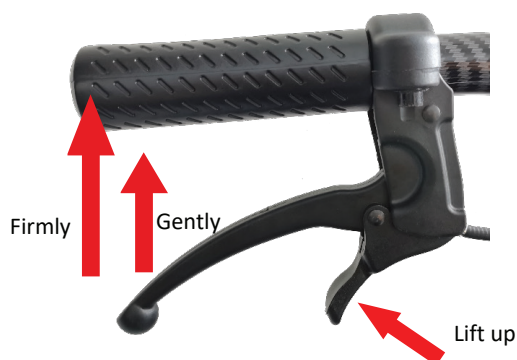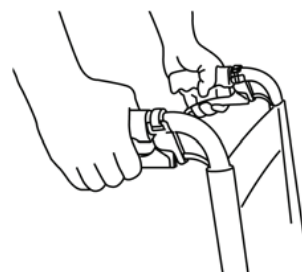

5.5 First Using Wheelchair

5.5.1 Operating the wheelchair by user.

5.5.1.1 Reading the instruction manual carefully, then assemble the wheelchair correctly and adjust the footrest, rear wheels and anti-tipping height properly.

5.5.1.2 Correctly assemble the cushions of backrest and seat.

5.5.1.3 Check that the brake and fold switch are locked.

5.5.1.4 Sit in the wheelchair, adjust your body position and gravity center.

5.5.1.5 Fasten the seat belt.

5.5.1.6 Please practice using the wheelchair in a spacious area and on a flat surface with accompanied.

5.5.1.7 Hold two sides of hand rims with both hands, push the hand rims forward and downward to move the wheelchair forward.

5.5.1.8 Hold two sides of hand rims with both hands, push the hand rims up and back to move the wheelchair backward.

5.5.1.9 Push the hand rim with your right hand to turn to the left. Push the hand rim with your left hand to turn to the right.

5.5.1.10 Practice braking.

5.5.1.11 Practice to sit in the wheelchair and go off the wheelchair.

5.5.2 Used by caretakers.

5.5.2.1 Reading the instruction manual carefully, then correctly assemble the wheelchair correctly and well adjust the footrest, rear wheels and anti-tipping height.

5.5.2.2 Correctly assemble the cushions of backrest and seat.

5.5.2.3 Check that the brake and fold switch are locked

5.5.2.4 Please make sure the user sit in the wheelchair with proper body position and gravity center.

5.5.2.5 Fasten the seat belt.

5.5.2.6 Practice to push the wheelchair forward and backward, practice braking and parking.

5.6 Moving The User In And Off The Wheelchair

5.6.1 Move the wheelchair as close as possible to the place you want to move, such as sofa, bed, etc.


5.6.2 Check that two sides of brake are locked.


5.6.3 Flip up the foot plates. Do not stand on the foot plates

5.6.4 Move in and out from wheelchair side.

5.6.5 For users with strong physical strength, it is possible to directly sit in the wheelchair and go off the wheelchair by themselves.

 **Caution:** If necessary, please get in and out of the wheelchair with assistance.

 **Caution:** When getting in and out of the wheelchair, please keep the brakes locked to avoid to fall over.

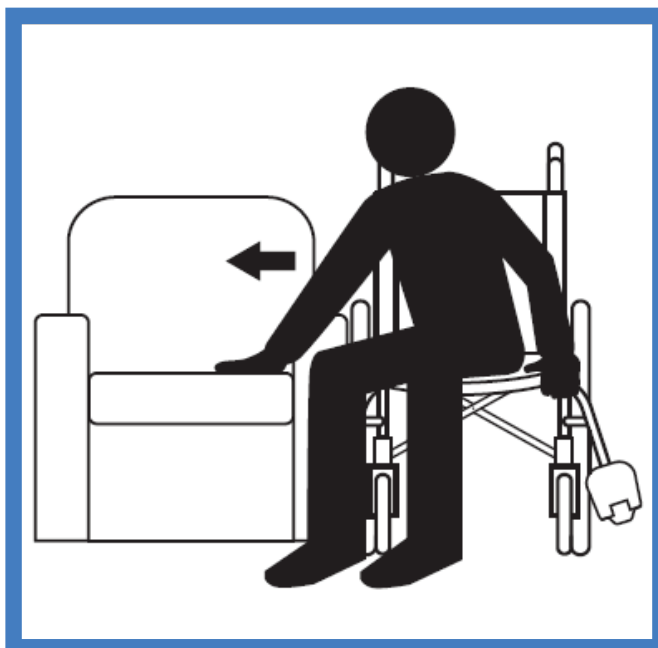

6.1 Cleaning The Wheelchair

6.1.1 Seat and backrest cushions can be washed with water and mild detergent once a week.

6.1.2 Other painted parts can be wiped with a cleaning sling or disinfecting sling once a week.

6.2 Disinfecting The Wheelchair

6.2.1 Wipe the armrests and handles with a cleaning sling or sanitizing sling dipped in 75% medical alcohol and dry them immediately, once a week is enough.



Caution!

- A.** Do not use high pressure cleaners to clean the wheelchair. Do not splash water on the wheelchair.
- B.** Do not use volatile chemical solvents, such as high-concentration alcohol, ether, high-concentration acid or banana water, gasoline, kerosene, etc. Also do not use abrasive waxes, sponges, brushes, etc.
- C.** All parts must be thoroughly dry before use after cleaning and disinfection.

6.3 Inspection, Service And Maintenance

6.3.1 Before using a wheelchair, you should check the following everyday.

- A.** After the wheelchair has been used for a period of time, it is necessary to check whether each component is loose.
- B.** Should add lubricating oil weekly.
- C.** After using the wheelchair, wipe the moisture and stains on the surface with a soft dry sling to prevent rust.
- D.** Seat and backrest cushions should always be kept clean to prevent skin infection.
- E.** The wheelchair should be placed in dry, ventilated, stable and non-corrosive environment.
- F.** Check that the frame can be folded smoothly.
- G.** Check that all the brakes are normal.
- H.** Check that the wheels turn smoothly and there is no shaking.

6.3.2 In order to better use the wheelchair and prolong the life of the wheelchair, we recommend that users should do regular inspections.

- A.** Weekly Inspection
Check the wheelchair looks clean, check the tires for cracks, and check the brakes are loose.
- B.** Monthly Inspection
Check whether the front and rear wheels are worn, if any wear, contact the dealer for replacement in time.
Check the rotation of the front and rear wheels.
Check the bearings of front and rear wheel.
- C.** Annual Inspection
The annual inspection is the same as above.

6.4 Reusing

If it needs to be reused after a long period of inactivity, please follow “6.2 and 6.3” above to clean, disinfect and maintain the wheelchair.

6.5 End Of Use



The symbol means that your product should be disposed of separately from household waste according the local law and regulations. At the end of the wheelchair life, take it to the designated recycling place. Sorting and recycling will help conserve natural resources, and ensure it is recycled in an environmentally friendly way.

Recycling of each part should be handled by a designated authorized recycling organization in accordance with local laws and regulations. As for plastic parts, frame and pipe parts, they can be handed over to professional recycling units for disposal after the end of use.

7.1 Repair And Replacement

The warranty period is one year from the date of sale.

Purchase from the designated dealer or the manufacture, operate and use in accordance with this instruction manual, and repair at the designated maintenance place during the warranty period. The manufacture promises to repair or replace different parts for product quality problems caused by materials or manufacturing.

7.2 Maintenance Parts And Period

From the date of manufacture

Repair the main frame within 3 years.

Consumable items (such as tires, grips, seats, back cushions, etc.) are not covered under warranty.

Under normal circumstances, consumers are responsible for the losses.

Quality problems caused by the following conditions do not have the effect of guarantee.

1. Operation not in accordance with the instructions manual and other human factors.
2. Not repaired at a company-designated place or using parts of other manufactures.
3. Expired
4. Caused by force majeure such as typhoons, floods, fires, earthquakes, wars, etc.

7.3 Maintenance Records

Maintenance Records			
Date	Issue	Caused	Remark

Name:	
Address:	Telephone:
Product Name:	Barcode:
Purchasing Date:	
Warranty Period: (Except for wearing parts), Others show in the instruction manual.	

NOTE: Please go to the dealer for warranty registration within one month, this card will be valid with the seal of the dealer or the manufacturer, and the warranty will be carried out with this card.

8.1 Transportation And Storage

- A.** Ambient Temperature Range: -13°F — 104°F;
- B.** Relative Humidity: $\leq 80\%$;
- C.** Atmospheric Pressure Range: 80kPa~105kPa;
- D.** Can be transported in general transportation.



journey solite[®] c2

If you have any questions, call us at our toll free number:

1-800-958-8324